Controllo D-Dimero / D-Dimer Control

Turbidimetria / Turbidimetry

REF. 6747 – 2x1.0 ml

AZIENDA CERTIFICATA DNV

UNI EN ISO 9001:2008
UN EN ISO 13485:2012

USO PREVISTO

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INTENDED USE

Is intended for use as quantitative quality controls for the D-Dimer assay only. It is a set of two control levels. These two levels are to be used exclusively with the D-Dimer kit (Ref. 6737).

REAGENTS - WORKING SOLUTIONS

Reactive Components

- Human serum and additives.

Non - Reactive Components

- Sodium azide (NaN₃) < 0.1%

The Controls are lyophilized (2x1ml). The concentration of the controls is lot specific and the D-Dimer concentrations are expressed in µg/ml FEU.

PREPARATION

Reconstitute each control with 1.0 ml of distilled water. Leave at room temperature for 30 minutes, inverting gently to dissolve contents. Mix carefully before each use.

STORAGE AND STABILITY

Store at 2-8°C. **DO NOT FREEZE.**

The reconstituted controls are stable 14 days at 2-8°C.

PRECAUTIONS AND WARNINGS

- For in vitro diagnostic use.
- Components from human origin have been tested and found negative for the presence of HbsAg, HCV and antibody to HIV 1/2. However handle cautiously as potentially infectious.
- Avoid contact with skin and eyes.
- Contains sodium azide, which may react with lead or copper plumbing to form explosive compounds.
- For more information refer to the Material Safety Data Sheet for this product.

MATERIAL REQUIRED AND NOT PROVIDED

Kit D_Dimer – REF. 6736 - 1x20 + 1x8 ml

Kit D-Dimer Calibrator – REF. 6746 - 5x1ml

REFERENCES

- BJK Guideline. British Journal of Haematology. 124, 15-25.
- Alan H.B. Wu. Tietz Clinical Guide to Laboratory Tests. Fourth Ed. Saunders Elsevier, 11830 Westline Industrial Drive, St Louis, Missouri 63146. 2006; 328-329.

LOT 102150

2017/03

Controllo D-Dimero	Value (µg/ml FEU)
Controllo 1	0.86±0.17
Controllo 2	3.72±0.74

D-Dimer Control	Value (µg/ml FEU)
Control 1	0.86±0.17
Control 2	3.72±0.74

Giesse Diagnostics srl

V. Enrico Fermi, 3 - Z.I.V. Tiburtina Km 18.300 - 00012 Guidonia Montecelio (RM) - Italia

Tel. +39 0774 051100 - Fax +39 0774 051111

e-mail: info@giesseiagnostics.com – web site: www.giesseiagnostics.com

674707

Ed. 2015/10 rev. 00