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INTENDED USE

Solution for extraordinary washing on reading cuvette of automatic analyzer: SAETTA200.^(*)

^(*) Name registered by GIESSE DIAGNOSTICS Srl

PRODUCTS CHARACTERISTICS

Liquid solution, ready to use.

PROCEDURE

Put the solution in place 01 of the instrument and do the cuvette's special washing at least once a week and/or when the analyzer give non sense results. At the end of washing procedure remove the solution from the Analyzer and keep it tightly closed.

STORAGE AND STABILITY

The product is stable until the expiration date indicated on the label when stored at 2-25°C, protected from light and in the absence of contamination.

Keep bottles closed when not in use.

PRECAUTIONS AND WARNINGS

EC Regulation 1272/2008 (CLP):

Warning

- H319** Causes serious eye irritation.
H315 Causes skin irritation.
H335 May cause respiratory irritation.
P264 Wash with water thoroughly after handling.
P280 Wear protective gloves/clothing; eye / face protection.
P304+P340 IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.
P312 Call a POISON CENTER or doctor/physician if you feel unwell.
P403+P233 Store in a well ventilated place. Keep container tightly closed.

Use the normal precautions required in the laboratory.

WASTE DISPOSAL

Dispose of product according to local laws.

Giesse Diagnostics srl

V. Enrico Fermi, 3 - Z.I. V. Tiburtina Km 18.300 - 00012 Guidonia Montecelio (RM) - Italia

Tel. +39 0774 051100 - Fax +39 0774 051111

e-mail: info@giessediagnostics.com - web site: www.giessediagnostics.com